



Osso Nurse Training Quick Start Guide

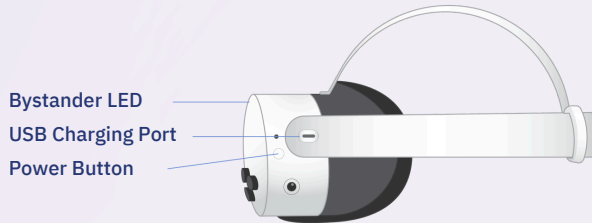

1 Charge and power on headset.



2 Adjust headset fit.

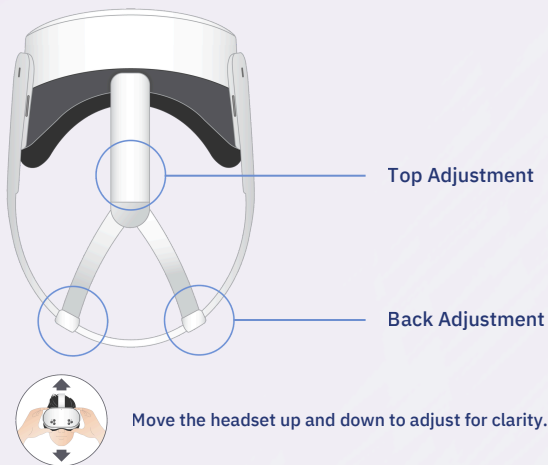

3 Get to know controllers.

Trigger Button
Interact with objects

Outside of Scenarios:
Select buttons and navigate menus

Grip Button
Grab object



Thumbstick

Interact with objects

Collaborative Training: Movement

Meta Quest Button (Hold 3s)

Reset view

Meta Quest Button (Quick Press)

Open Home Screen Toolbar

4 Set up a boundary.

1. Put on the headset.
2. When prompted select **Draw a Roomscale** boundary.
3. Look down and confirm the floor height. Adjust by following on-screen prompts.
4. Draw the Roomscale boundary with the trigger button. Make sure the lines connect.
5. Select **Confirm** when done.

Tip: If the space is too small, a **Stationary** boundary may be prompted. **Roomscale** is always recommended.




5 Set up WiFi.

1. Go to the **Quick Settings** menu by selecting the WiFi icon located next to the clock on the toolbar.



2. Select the WiFi tile and enter Network information.

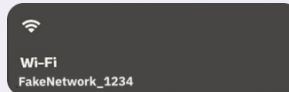

6 Log in and open app.

1. On any browser, go to loop.ossovr.com and log in.
2. Select a **Scenario** and complete **Prebriefing** until the scan **QR Code** screen appears.
3. Open the Osso Nurse Training app on the headset, accept any permissions if prompted, and point it at the QR Code to begin.

7 Launch a scenario mode.

When prompted in the headset, select a mode:

- **Practice Mode:** Includes hints and prompts.
- **Test Mode:** No hints by default; press Y for a hint.
- **Collaborative Training:** Multiple users join a session.

*Action not available in all scenarios.

©2026 Osso VR, Inc. All rights reserved. "Osso VR name, Osso VR logo, and other Osso VR trademarks are property of Osso VR, Inc. All other product and company names are trademarks or registered trademarks of their prospective holders, and use of them does not imply any affiliation with or endorsement by them.